


Microbiology Testing for HPV (Human Papillomavirus)* and Co-collection w/ PAP Testing, Chlamydia trachomatis, Neisseria gonorrhoeae, & Trichomonas vaginalis

*FDA approved for primary HPV testing and reports High-Risk HPV genotypes

*With one specimen all
testing can be performed.*

Epic Orderable Test Names = Pap Test, HPV High Risk PCR Testing (Both orders allow for co-testing)

ACCEPTABLE SPECIMEN: ThinPrep® Liquid-Based system

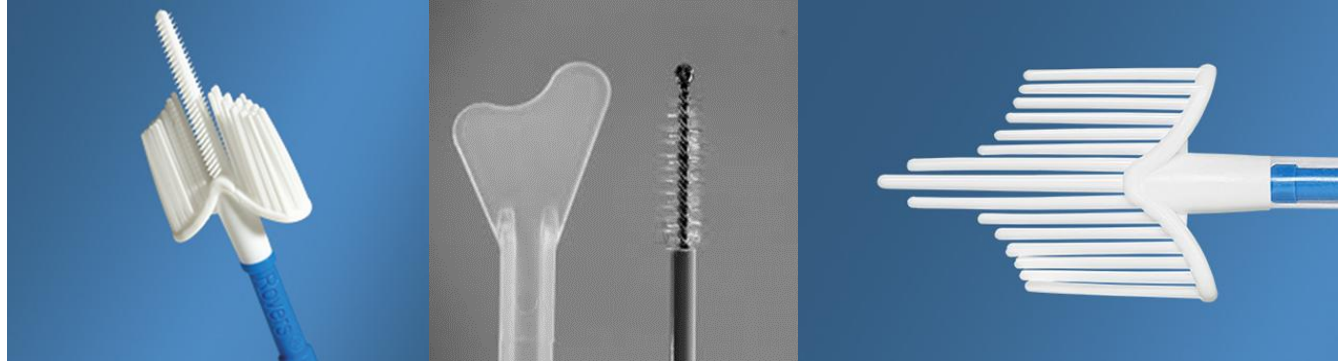

(ThinPrep® Can be collected with (in order pictured above): Rovers Cervix-Brush Combi **OR** Cytobrush/Plastic Spatula **OR** Broom-like Device).

Workday: #218879

#218878

#218877

Note: The Rovers Cervix-Brush Combi device should not be used during pregnancy.

SOURCE AND BODY SITES: Sources = Cervix or Vagina

COLLECTION AND TRANSPORT: Transport ThinPrep vial to the laboratory at room temperature within 24 hours.

For questions regarding specimen collection and handling, contact **Cytology (Pap)** at 628-2591; **Microbiology (HPV & STIs)** at 628-4000.