






Microbiology Testing for Sexually Transmitted Disease (STD)

<p>Order:</p>	<p><i>Chlamydia trachomatis</i>/ <i>Gonococcus (Neisseria gonorrhoeae)</i> by DNA Amplification</p> <p>Trichomonas Direct testing-dedicated tube</p>	<p><i>Chlamydia trachomatis</i>/ <i>Gonococcus (Neisseria gonorrhoeae)</i> by DNA Amplification</p> <p>Trichomonas Direct Testing</p>	<p><i>Gonococcus (Neisseria gonorrhoeae)</i> Culture</p>	<p><i>Chlamydia trachomatis</i> Culture</p>
<p>Specimen:</p>	<p>Female Endocervix</p>	<p>Female Urine Male Urine First void, 20-50 mL collected into sterile container, using pipette provided in kit add 7 mL of urine into transport tube</p>	<p>Eye Genital Respiratory, Upper Rectal</p>	<p>Eye Genital – Penis Genital - Vagina Genital - Vulva Rectal Swab Respiratory, Upper</p>
<p>Collection Container:</p>	<p>Xpert® Vaginal/ Endocervical specimen collection kit (pink lid) Workday # 129641</p> 	<p>Xpert® Urine collection kit (yellow lid) Workday # 39973</p> 	<p>Aerobic/Anaerobic transport system (e-swab) Workday # 78</p> 	<p>Universal viral transport media (UTM, red lid) Workday # 39659</p> 
<p>Rectal- refer to next page</p>		<p>VCUHealth 01/22/25 AH</p>		

Chlamydia and Gonococcus, NAA
Collection Device utilized by LabCorp

Epic Order: Miscellaneous Test

Order Question	
Test code:	188672 (Enter only the reference laboratory test #; Labcorp: 6 numerical digits)
Test Name:	Chlamydia/Gonococcus, Rectal Swab , NAA
Testing Laboratory:	Click LABCORP
Collection Container:	APTIMA® Unisex Swab
Special Handling:	Maintain specimen at room temperature or refrigerate.

Specimen Type	Collection Kit	Handling/Transport
Rectal	<p>APTIMA® Unisex Swab Transport</p> 	<p>Store/Transport at room temperature.</p>